



USER INSTRUCTIONS

MediBoard

The PC keyboard for medical technology



MediBoard with mousepad

REF	H+K Nr. 67802xxx	(L 410 x W 220 x H 26 mm)
------------	------------------	---------------------------

MediBoard w/o mousepad

REF	H+K Nr. 67801xxx	(L 410 x W 220 x H 26 mm)
------------	------------------	---------------------------

The MediBoard features the same characteristics of a generic computer keyboard, however, it is specially designed for application at hospitals and medical offices. The fully sealed surface is suitable for multiple daily cleaning with standard disinfectants. The keyboard has an antimicrobial film surface, which prevents the spread of various viruses and bacteria. It can be connected with other devices through a USB port.

CLEANING

- ☉ We recommend mild detergent or soap. Dry with soft cloth.
- ☉ To disinfect, surface disinfectant should be used consisting of concentrates and water. A list of tested disinfectants can be found under the downloads tab at www.keywi-usa.com.

	Products may not be thermally disinfected
	Abrasive cleaning aids and solutions containing alcohol are not permitted
	0° C to +50° C
	2012
	Hoffmann + Krippner GmbH, Siemensstraße 1, 74722 Buchen

The recommendations and references for proper cleaning of our products are based on internal and external studies. We are not liable for any damages due to improper handling and use.

INDICATION

Suitable to fulfill strict hygiene standards and infection control requirements in medical environments with high risks of infection (please refer to Infection Prevention by the Commission for Hospital Hygiene at the Robert Koch Institute).

CONTRA-INDICATION

Without inspection not suitable to connect to medical devices above risk category IIa.

PRODUCT RETURNS

Products can only be returned if declared as "clinically safe".

SYMBOLS

SN	Serial number
REF	Product Number
	Manufacturer
	Caution, Safety Information
	Date of Manufacture
	Temperature Resistance
	Refer to User Manual

